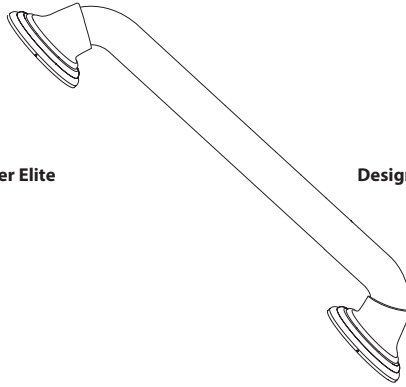
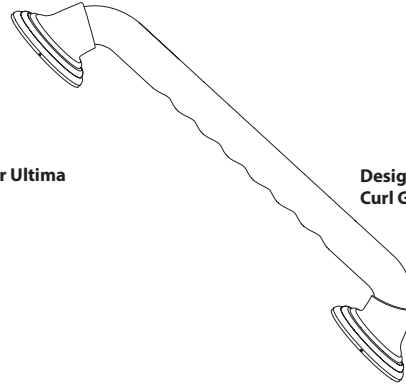
Designer Elite

Designer Ultima

Designer Ultima with Curl Grip

Thank you for purchasing this Home Care® by Moen® product. Home Care by Moen is a leader in grab bars, offering top-quality products that are attractive and innovative, with performance features like a 500 lb. weight pull capacity that exceeds ADA requirements.

The entire Home Care by Moen family of products offers people new options for independent living by providing outstanding comfort, convenience and style.

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
**All Home Care by Moen grab bars feature the exclusive patented SecureMount™ mounting flange.**

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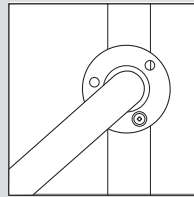**Conventional Mounting / Montage conventionnel / Montage convencional**


Conventional mounting configurations often only allow for one or two screw connections to the stud, making installation difficult and less secure.

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**SecureMount™ / SecureMount™ / SecureMount™**

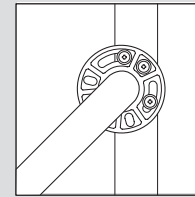The patented SecureMount™ design makes grab bar installation easier and more secure with an innovative design that features multiple slots for screws, allowing secure connection with three screws into the wood stud at any angle.

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
**Components included / Pièces incluses / Componentes incluidos**

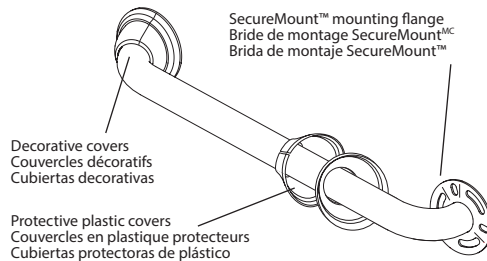
**Grab bar, with SecureMount™ mounting flanges and decorative covers**

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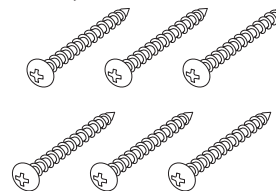
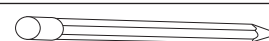
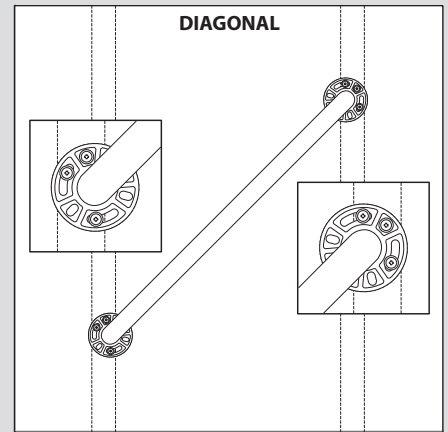
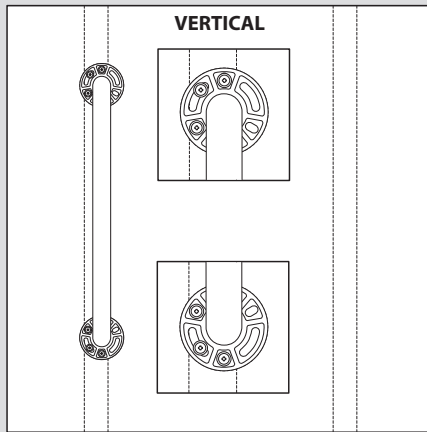
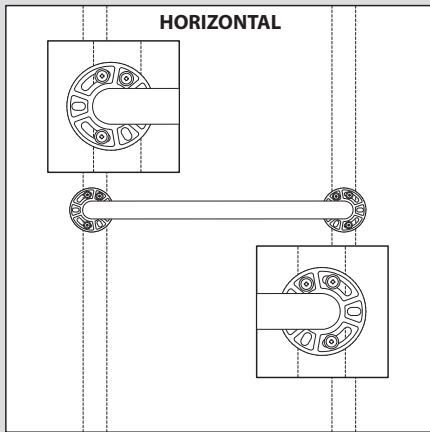
## Instructions / Instrucciones / Directives

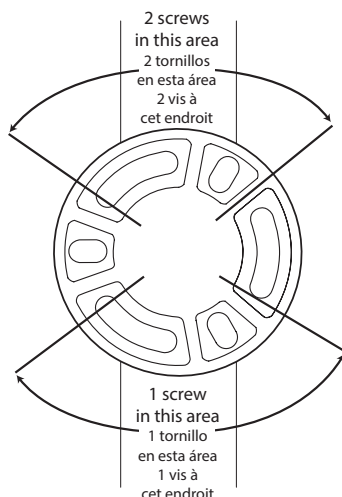- 1). Please note that screws **MUST** be mounted into a wood stud for secure installation.
- 2). Slide the decorative cover away from the hub end to expose the SecureMount™ mounting flange. Do not remove protective plastic from decorative cover until indicated later in these instructions.
- 3). Locate the studs behind the wall by using a stud sensor or other stud locating method. Studs are usually 16 in. apart; locate the center of each stud.
- 4). Place the grab bar on the wall where desired so that the SecureMount™ mounting flange is positioned over a wood stud. Use a level to verify desired placement. Decide whether you would like to position your grab bar horizontally, vertically or at an angle, remembering that **BOTH** ends of the grab bar **MUST** be positioned over a wood stud. Note: Diagonal/angled installation is possible only with grab bars 24 in. or longer.

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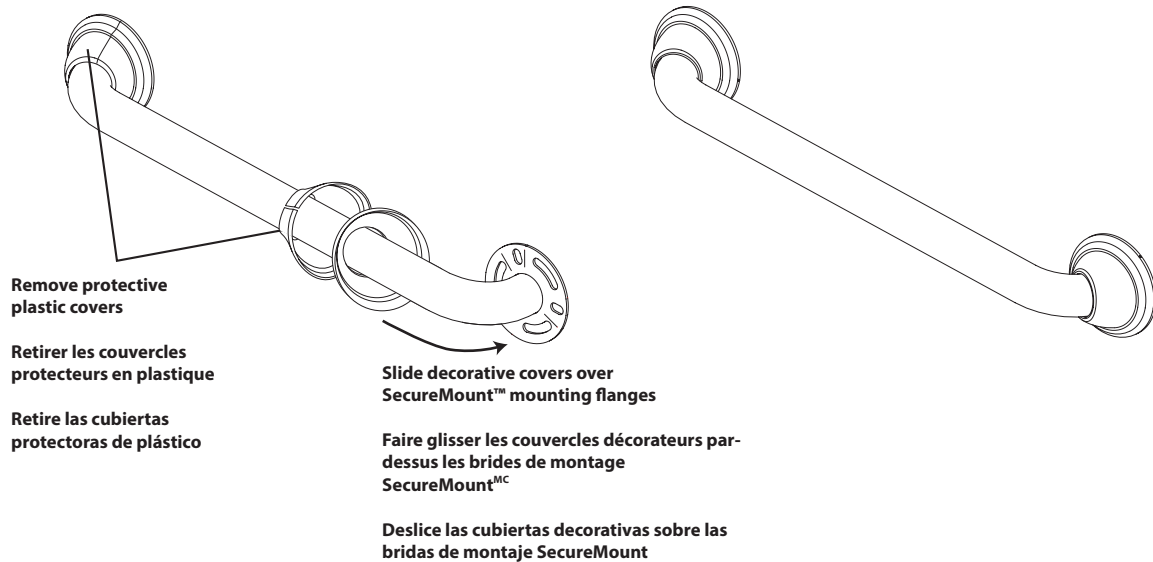
## MOEN® HOME CARE® GRAB BAR INSTALLATION INSTRUCTIONS FOR DESIGNER ELITE, DESIGNER ULTIMA AND DESIGNER ULTIMA WITH CURL GRIP GRAB BARS

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### WARNING:

TO PREVENT SERIOUS OR FATAL INJURY: FOR USE ONLY BY PERSONS WEIGHING LESS THAN 500 POUNDS. Consult a healthcare professional for advice and instructions specific to your needs before use. Contact Moen Home Care immediately at 1-800-289-6636 if any components are missing or damaged. Install in accordance with Moen Home Care instructions. Do not modify this product in any way. FOR QUESTIONS OR ASSISTANCE WITH INSTALLATION CONTACT MOEN HOME CARE AT 1-800-289-6636.

### LIMITED LIFETIME WARRANTY:

If this product fails due to materials or workmanship at any time during the life of the product, Moen Home Care, a division of Moen Incorporated, will replace it free of charge, postage-paid. Simply contact Moen Home Care at 1-800-289-6636 for replacement information. This warranty does not cover products which have been abused, altered, damaged, misused, cut or worn. TO THE EXTENT PERMITTED BY LAW, MOEN DISCLAIMS ALL OTHER IMPLIED OR EXPRESS WARRANTIES INCLUDING ALL WARRANTIES OF MERCHANTABILITY AND/OR FITNESS FOR A PARTICULAR PURPOSE.

**LIMITATION OF LIABILITY:** THIS WARRANTY IS YOUR SOLE REMEDY AND MOEN SHALL NOT BE LIABLE FOR ANY DAMAGES, WHETHER DIRECT, INDIRECT, INCIDENTAL, SPECIAL, CONSEQUENTIAL, EXEMPLARY, OR OTHERWISE, INCLUDING LOST REVENUES AND LOST PROFITS, ARISING OUT OF ANY THEORY OR RECOVERY, INCLUDING STATUTORY, CONTRACT OR TORT. NOTWITHSTANDING THE TERM OF ANY LIMITED OR IMPLIED WARRANTY, OR IN THE EVENT THAT ANY LIMITED WARRANTY FAILS OF ITS ESSENTIAL PURPOSE, IN NO EVENT WILL MOEN HOME CARE'S ENTIRE LIABILITY EXCEED THE PURCHASE PRICE OF THIS PRODUCT. SOME STATES AND PROVINCES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES SO THE ABOVE LIMITATIONS OR EXCLUSIONS MAY NOT APPLY TO YOU. THIS LIMITED WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE RIGHTS WHICH VARY FROM STATE TO STATE AND PROVINCE TO PROVINCE.

For additional warranty information, visit [www.moen.com/homecare](http://www.moen.com/homecare)  
Meets or exceeds applicable requirements for the Americans with Disabilities Act

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25300 Al Moen Drive • North Olmsted, OH 44070-8022 • [www.moen.com/homecare](http://www.moen.com/homecare)  
Moen Home Care is a trademark of Moen Incorporated ©2018

Moen Incorporated. © 2018 / Moen incorporée. © 2018 / Moen Incorporated. © 2018